

PainPump[®]2 BlockAid


Resetting the Pump

WARNING: Only trained and experienced health care professionals should access the code and edit settings. Failure to comply may result in patient injury.


- After the settings are edited, a bolus dose cannot be activated until the new lockout interval countdown reaches zero. Even when the lockout interval is not changed, the unit begins to count down the interval from its original starting point.
- After the pump is locked, you must use a security code to unlock and edit the settings. The security code is embedded in the numerical information currently displayed on the screen.

Examples:


The security code is comprised of the 4 digits displayed in the second column of numbers from the right as shown below.

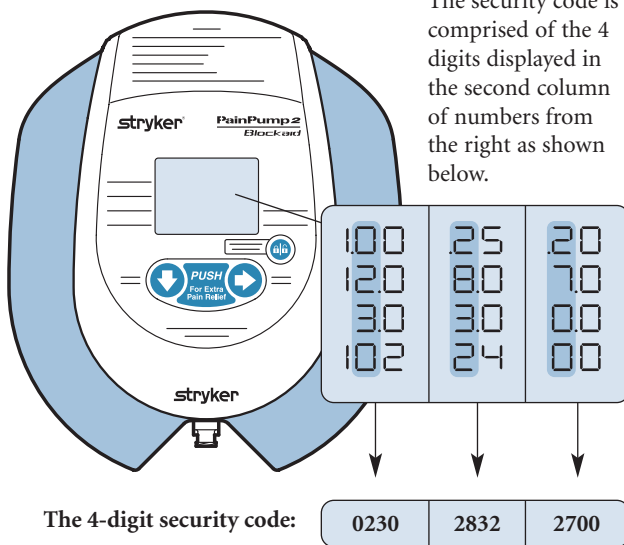

- Obtain the 4-digit security code from the information screen as explained above.
- Push the Lock button . The CODE screen will appear.

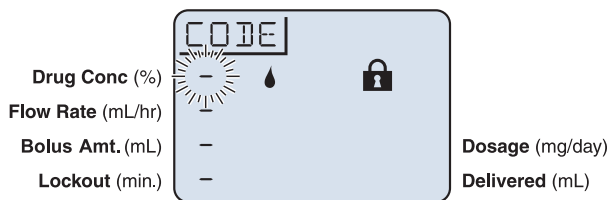


- Use the Select Settings and Adjust Value buttons to enter the 4-digit code.

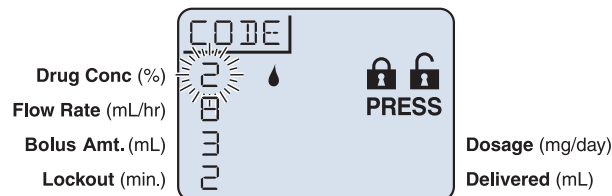


- Push the Lock Button . If code was entered correctly, the EDIT screen will appear.

Note: NO is displayed on the screen if the security code is entered incorrectly. If the security code is entered incorrectly 5 consecutive times within 30 minutes, the screen will display WAIT and prevent access to the CODE screen for 30 minutes. Call the Stryker 24-hour hotline if you have questions at 877-848-5756.


- Use the Select Settings and Adjust Value buttons to edit the settings.

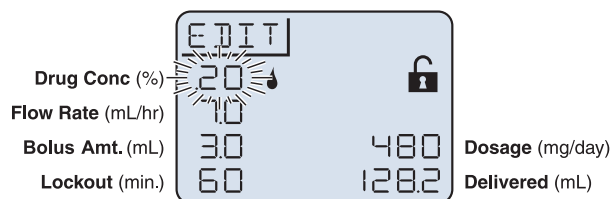


- Press the Lock button once. The PRESS lock symbol will appear.

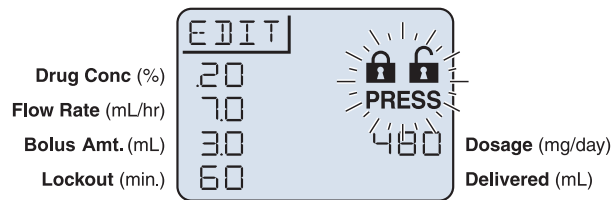

- Press the Lock button again to lock the modified settings. The screen will display OK when the unit locks and the unit will begin using the modified settings.

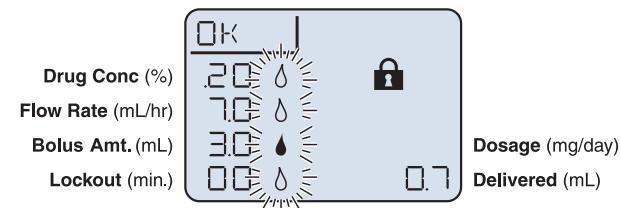

Note: The screen will display EMPTY once all the medication from the reservoir has been dispensed.

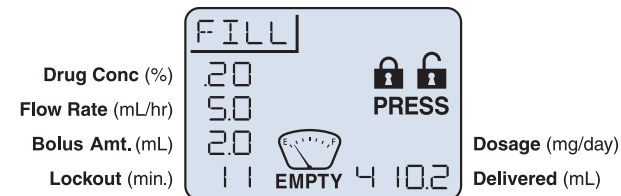

This indicates that the pump should be refilled or discontinued. If refilling, please refer to the REFILL instructions on the back of the pump.

Note: The screen will display DONE once the pump has dispensed 1000mL of medication. This indicates that the pump should be discontinued.



To turn the pump off at anytime, press and hold the Lock button. After 2 seconds you will see HOLD displayed on screen and after 5 seconds the unit will turn off.

